

NATROX™

Oxygen Wound Therapy

Healing in the palm of your hand

NATROX™ works by supplying a continuous flow of pure humidified oxygen to the skin and wound bed under the dressing (12-14ml/hour). When combined with an appropriate occlusive dressing, an oxygen-enriched atmosphere is maintained around the wound area, helping to accelerate the wound healing process.



- Full mobility during Oxygen Therapy, comfortable and silent.
- Strong evidence of pain reduction.
- ODS may be cut to suit wound size and shape.
- Rechargeable batteries and long lasting technology provide low cost treatment.
- Clinicians own choice of dressings to manage exudate.

How to use

- Place ODS in the centre directly on wound surface.
- The ODS may be trimmed to suit small wounds.
- Cover the ODS and wound with a highly absorbent layer.
- Use a semi or occlusive film above the absorbent layer only if the absorbent layer does not have one.
- ODS is designed to be used with compression.
- The NATROX™ system can be used with most wound types.
- Fit a charged battery to NATROX™ and connect the ODS.
- A green light flashes to indicate oxygen flow.
- Two batteries are provided, change the battery every day.
- Replace ODS with each dressing change.

