

NATROX™

Oxygen Wound Therapy

QUICK START GUIDE



Continuous Diffusion of Oxygen Therapy
delivery system

Oxygen Generator (OG) and
Oxygen Delivery System (ODS)

Inotec AMD^{LTD}

This guide is designed for patient use and must be read in conjunction with the full *Instructions For Use* enclosed with the NATROX™ Continuous Diffusion of Oxygen Therapy delivery system.

Overview

The dressings used to protect your wound also restrict natural oxygen in the air. NATROX™ is designed to constantly refresh your wound with a low flow of pure humidified oxygen providing the energy source to help regenerate new tissue.

Oxygen Generator (OG) and Oxygen Delivery System (ODS)

- Operating costs are very low as rechargeable batteries are used - coloured **Green & Blue**.
- A mains charger is supplied.
- A fully charged battery powers the OG for approximately 30 hours.
- One battery must **ALWAYS** remain on charge while the other is used in the OG.

When charging a yellow light flashes and stays **ON** when fully charged. Batteries are designed to be left on constant charge, no damage will be done and you will always have a fully charged battery each morning to power your NATROX™.



Operating instructions

- Fit the charger to a wall socket **where it can remain undisturbed**.
- Plug the charger into **Blue** battery and see the yellow charge light flash. Leave on charge. Always charge batteries before first use or after storage.
- Now fit the **Green** battery to the OG and see the small green light flash in the OG. Oxygen is now flowing and the OG is ready for connection to the ODS and for oxygen therapy to begin.



Battery Charging and Changeover

- A battery must always be left on constant charge. Next morning check that the yellow charge light is **ON** showing a fully charged battery.
- Remove **Blue** battery from the charger by holding and pulling the plug and fit this to the OG.
- Now place **Green** battery on charge. Continue this routine. A simple chart or diary may help you to remember to change over correctly, for example Monday **Green**, Tuesday **Blue**, Wednesday **Green** and so on.

If you have any questions, discuss with your clinician and do remember to read the full version of *Instructions for Use*.

NATROX™

Oxygen Wound Therapy

NATROX™ OG



NATROX™ ODS



Additional information can be found on our website at www.inotecamd.com

For questions or comments on this device or on the information supplied in this instruction booklet, please call Inotec AMD on +44 (0)1763 209007

Inotec AMD Ltd
Butts Business Centre, Fowlmere,
Cambridge, SG8 7SL, UK

Tel: +44 (0)1763 209007
Email: info@inotecamd.com

www.inotecamd.com

NATROX™ QSG English V5

Inotec AMD LTD